## PRODUCTS FOR STRIPPING IN ORTHODONTICS

WE THANK FOR THE CLINICAL PHOTOGRAPHS GRANTED:

- DR. M. CIARAVOLO, NAPOLI, ITALY
- Dr. F. Garino, Torino, Italy
- DR. G. FORTINI, FIRENZE, ITALY
- DR. E. CARLI, MASSA CARRARA, ITALY
- DR. A. CABURLOTTO, VENEZIA, ITALY
- DR. F. GIUNTOLI, PISTOIA, ITALY

manufactured by



#### **INTENSIV SA**

Via Molino 107 6926 Montagnola, Switzerland Phone +41 91 986 50 50 Fax +41 91 986 50 59 info@intensiv.ch DISTRIBLITED IN FLIROPE BY



#### **LEONE S.p.A.**

Via P. a Quaracchi 50 50019 Sesto Fiorentino, Firenze, Italy Phone +39 055 30 44 1 Fax +39 055 37 48 08 info@leone.it

www.leone.it

#### **SALES DEPT.**

Phone +39 055 30 44 620 Fax +39 055 30 44 05 export@leone.it



# PRODUCTS FOR STRIPPING IN ORTHODONTICS







## **STRIPPING PROCEDURE**

Uni/bilateral reduction of interproximal enamel is a procedure that boasts many years of clinical use. It is very useful to gain space and limit the need for extractions, in both vestibular and lingual orthodontic treatments, while it is often included in the therapy with aligners.

The Intensiv product line dedicated to **IPR (InterProximal Reduction) Protocols** allows you to carry out fast and safe treatments. The procedure is calibrated, quick, and comfortable for the patient.

Basing upon a selection of products, the Stripping protocol in Orthodontics was developed to perform the opening of contact points, the reduction of proximal surfaces, the control and verification of the gained space between the teeth and finally, restoration and polishing of treated surfaces.

START		PROTOCOL USAC IPR TIME		END	
		$0,1  \text{mm} \longrightarrow 5'' \text{approx}$ $0,2  \text{mm} \longrightarrow 10' \text{approx}$	, , , , , , , , , , , , , , , , , , ,	CONTOURING	
OPENER	INCONSTRUCTS	$0,3_{\text{mm}} \longrightarrow 3''_{\text{approx}}$ $0,4_{\text{mm}} \longrightarrow 5''_{\text{approx}}$	and the second of the second o	700	
		$0,5 \text{mm} \longrightarrow 7''$		POLISHING	
OPENING CONTACT POINTS		ENAMEL REDUCTION AND SPACE GAIN BETWEEN THE TEET	VERIFICATION H OF GAINED SPACE	FINISHING AND POLISHING	

<sup>\*</sup>Usage time of the Ortho-Strips is indicative and depends on the pressure exerted by the operator, the amount of irrigation used and the wear of the Ortho-Strips.

## **IPR-DistanceControl**

Hand instrument for verification and checking of the space created between the teeth during the Stripping procedure. Available as single packs or Sets of 8.

Ref. LIPRDC010	thickness 0,10 mm
Ref. LIPRDC015	thickness 0,15 mm
Ref. LIPRDC020	thickness 0,20 mm
Ref. LIPRDC025	thickness 0,25 mm
Ref. LIPRDC030	thickness 0,30 mm
Ref. LIPRDC040	thickness 0,40 mm
Ref. LIPRDC050	thickness 0,50 mm
Ref. LIPRDC100	thickness 1,0 mm

Ref. **LIPR-DC-SET** Set with assorted 8 thick sizes



#### **BENEFITS**

- Durable stainless steel
- Secure application due to the handle
- Large number of measurable distances
- To check further thickness sizes, combine the single IPR-DistanceControl
- The elements can be used individually or grouped with a rivet








## MECHANICAL STRIPPING


## Intensiv Ortho-Strips, Central



Innovative oscillating diamond Strips in the central operating area that allow safe treatment avoiding the creation of steps and abrasion of dentin. The slightly bendable bow allows the Strip to adapt to the horizontal tooth anatomy and to opposing tooth surfaces. Ideal for reduction, finishing and polishing of approximal enamel surfaces in Orthodontics.

During the process of reduction, finishing and polishing of enamel in Orthodontics (Stripping) it is possible to create a cervical step or to unintentionally remove dentin below the equatorial area of the tooth, may causing subsequent patient dental sensitivity.

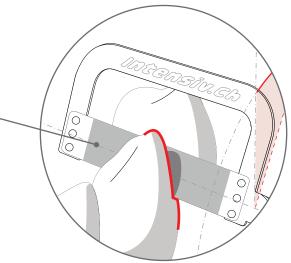

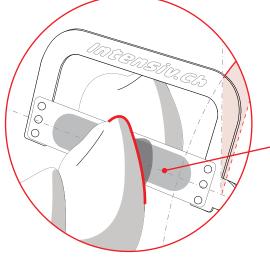

Thanks to the diamond coating in the central operating area, Intensiv Ortho-Strips System, Central allows for high precision in the approximal reduction, finishing and polishing of enamel, respecting the original morphology and dental tissues.

#### **BENEFITS**

- Easy introduction into the interdental contact points
- Prevention of cervical steps
- Dentine hypersensitivity would not be supported
- Conservation of the original morphology of the tooth equator
- Sterilisable



CLINICAL USE OF ORTHO-STRIPS CENTRAL



## Intensiv Ortho-Strips, Central

Available in 4 different grit sizes, in the Double-Sided diamond version on both sides, and One-Sided diamond on one side, in the variant "L", left (left side) and "R", right (right side) for the mesial and distal sides in the upper and lower jaws.

- Strip thickness of the not diamond-coated area: 0.05 mm
- Total height of Strip: 3.7 mm
- Diamond area height: 2.7 mm
- Diamond area length: 13 mm









#### Ref. LOST400Set02/6



## Tray Ortho-Strips, Central

Stainless steel tray with six colored silicone holders.


#### **CONTENT:**

- 2 Ortho-Strips, Double-Sided
- Ref. LOS08OP, Ortho-Strips, Opener
  - Ref. LOS80C, Ortho-Strips, Extracoarse
- 4 Ortho-Strips, Central Double-Sided
- Ref. LOS60C-CEN-DS, Ortho-Strips, Central Coarse
- Ref. LOS40M-CEN-DS, Ortho-Strips, Central Medium
- Ref. LOS25F-CEN-DS, Ortho-Strips, Central Contouring
- Ref. LOS15POL-CEN-DS, Ortho-Strips, Central Polishing

## MECHANICAL STRIPPING

Ref. L6000/WG-69A without light Ref. L6000/WG-69LT with light

### Intensiv IPR Set

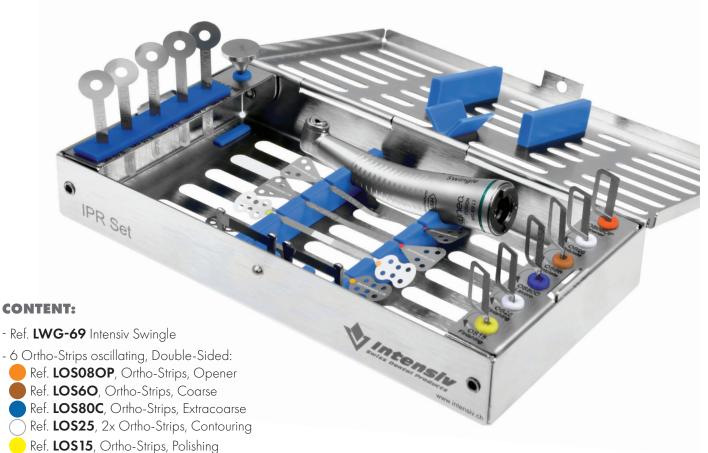

- 5 IPR-DistanceControl:

- Ref. LIPRDC010 (thickness 0,10 mm)

- Ref. **LIPRDC015** (thickness 0,15 mm)

- Ref. LIPRDC020 (thickness 0,20 mm)

- Ref. **LIPRDC030** (thickness 0,30 mm)

- Ref. LIPRDC050 (thickness 0,50 mm)

- 4 Ortho-Strips manual

- Ref. **LOSM8040**, 2x Extracoarse/Medium grit size

- Ref. **LOSM1508**, 2x Extra/Ultrafine grit size

- 2 ApproxOpener

- Ref. LAO2018OS, One-Sided

- Ref. LAO2018DS, Double-Sided

- Ref. L053 Ejector Intensiv Swingle

The tray includes all the instruments for stripping arranged in an ergonomic and practical way for the treatment of each patient.

Composed of a base and a lid joined by a hinge, it is made of polished stainless steel with suitable dimensions for autoclave sterilization.

Ortho-Strips and space gauges are placed in two internal folding holders with the part numbers of the instruments and the color reference. These holders must be oriented parallel to the working bench during use of the instruments and folded inside after sanitizing the contents of the tray, which will be ready for sterilization.

#### Ref. **L071**



## Refill for IPR Set

**CONTENT:** 6 Double-Sided Ortho-Strips

Ref. **LOS08OP**, Ortho-Strips, Opener

Ref. **LOS6O**, Ortho-Strips, Coarse

Ref. **LOS80C**, Ortho-Strips, Extracoarse Ref. **LOS25**, 2x Ortho-Strips, Contouring

Ref. LOS15, Ortho-Strips, Polishing

Ref. **LWG-69 A** without light Ref. **LWG-69 LT** with light

## Intensiv Swingle







#### **FEATURES:**

- 2 applications with only one contra-angle with micro mechanism, activated thanks to the toggle switch to choose between free axial rotation or fixed position of Intensiv Ortho-Strips System
- Contra-angle with integrated water spray system and light
- Reciprocating movement of 0,9 mm; 20.000 oscillating movements with  $40.000 \ \text{rpm}$
- 12 fixed axial positions (at each 30° within 360°)
- Free movement for positioning of Ortho-Strips in the desired position
- 2 years warranty
- Sterilisable



- Ref. **LWG-69**, Swingle
- Ref. **L053**, Swingle ejector
- Ref. LO54, Swingle cleaner
- Ref. **LOS40**, Ortho-Strips, Medium



Ref. **L053** 

## Intensiv Swingle **Ejector**





To facilitate the release of Ortho-Strips from the contra-angle.

## MECHANICAL STRIPPING

## Intensiv Ortho-Strips

Double-sided oscillating diamond coated Strips for reduction, finishing and polishing of approximal enamel in Orthodontics (IPR). Calibrated use according to the need for diagnosed space necessary in the treatment with aligners, brackets, or removable devices. Available single or into a set. Clean and sterilize Ortho-Strips thoroughly after each use to ensure a longer shelf life.

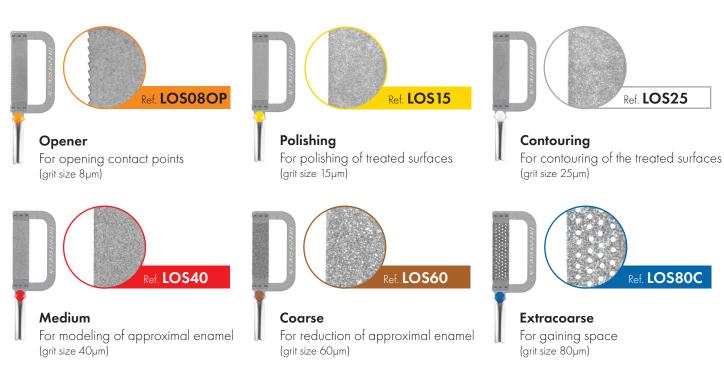

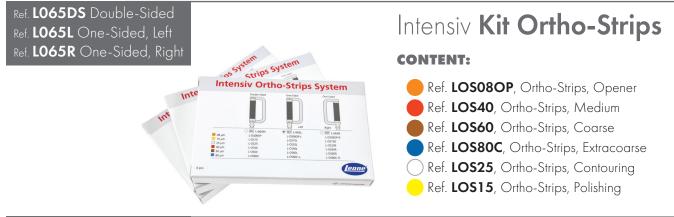



## Intensiv Tray Ortho-Strips

Compact stainless-steel tray with 6 silicone-colored holders for storage and protection of oscillating Ortho-Strips.

Ref. **L056ASS/6** 



## Silicone holders refills

6 pcs in assorted colors.

## Intensiv ApproxOpener

Designed to allow manual opening of extremely tight contact points: the special edge shape and the thickness of only  $8\mu m$  favor the precision of the introduction of the blade. Double-sided or one-sided diamond coating, the choice between the two types of Instruments is based on the need for unilateral treatment of the tooth to be reduced or the treatment of both adjacent interproximal surfaces.



#### Ref. **LAO2018OS**

- **Ultrafine** (grit size 8µm)
- one-sided
   diamond coating



#### Ref. LAO2018DS

- **Ultrafine** (grit size 8µm)
- double-sided diamond coating



## Manual Ortho-Strips

Stips perforated, with two grit sizes, ideal for reduction and finishing of interproximal surfaces. One-sided diamond coating.

Sterilisable. Reusable.



#### Ref. **LOSM1508**

For finishing and polishing of treated surfaces. One-sided diamond coating.

Available with two types of grit sizes:

- Polishing (yellow, grit size 15μm)
  - Opener (orange, grit size 8µm)



#### **BENEFITS**


- Efficient polishing of interproximal enamel
- Shaped grips for better manual grip during treatment

#### Ref. LOSM8040

For manual space creation.
One-sided diamond coating.

Available with two types of grit sizes:

- **Extracoarse** (blue, grit size 80μm)
  - Medium (red, grit size 40µm)

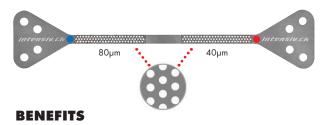

- Clutch-free treatment thanks to perforations of the working part
- Consistent reduction of interproximal enamel
- Triangular hand-grips for better manual holding during treatment

## **PRODUCTS PRICELIST**

REF. DESCRIPTION DIAMOND QUANTITY PRICES GRIT-SIZE PACK IN EURO\*

### **MECHANICAL INSTRUMENTS**

·III.	LIPR 6000/WG-69LT	IPR Set, Tray with contra-angle with light	19 pcs	1.825,00
	LIPR 6000/WG-69A	IPR Set, Tray with contra-angle without light	19 pcs	1.455,00
	LIPR 6000	IPR Empty Tray	1 pc	285,00
	LWG-69 LT	Intensiv Swingle, with light	1 pc	1.170,00
	LWG-69 A	Intensiv Swingle, without light	1 pc	790,00
/2"	L053	Intensiv Swingle ejector	1 pc	30,40
and the same	L054	Intensiv Swingle spray canal cleaner needle	1 pc	14,25

#### **ORTHO-STRIPS**


<u> </u>	LOS08OP/3	Ortho-Strips, Double-Sided, Opener	8µm	3 pcs	84,60
100	LOS08OP-L/3	Ortho-Strips, One-Sided left, Opener	8µm	3 pcs	84,60
A STATE OF THE PARTY OF THE PAR	LOS08OP-R/3	Ortho-Strips, One-Sided right, Opener	8µm	3 pcs	84,60
	LO\$15/3	Ortho-Strips, Double-Sided, Polishing	15µm	3 pcs	84,60
1550	LO\$15L/3	Ortho-Strips, One-Sided left, Polishing	15µm	3 pcs	84,60
	LOS15R/3	Ortho-Strips, One-Sided right, Polishing	15µm	3 pcs	84,60
A	LO\$25/3	Ortho-Strips, Double-Sided, Contouring	25 µm	3 pcs	84,60
	LOS25L/3	Ortho-Strips, One-Sided left, Contouring	25µm	3 pcs	84,60
	LOS25R/3	Ortho-Strips, One-Sided right, Contouring	25µm	3 pcs	84,60
A	LOS40/3	Ortho-Strips, Double-Sided, Medium	40µm	3 pcs	84,60
550	LOS40L/3	Ortho-Strips, One-Sided left, Medium	40µm	3 pcs	84,60
CELL	LOS40R/3	Ortho-Strips, One-Sided right, Medium	40µm	3 pcs	84,60
A	LOS60/3	Ortho-Strips, Double-Sided, Coarse	60µm	3 pcs	84,60
	LOS60L/3	Ortho-Strips, One-Sided left, Coarse	60µm	3 pcs	84,60
	LOS60R/3	Ortho-Strips, One-Sided right, Coarse	60µm	3 pcs	84,60
	LOS80C/3	Ortho-Strips, Double-Sided, Extracoarse	80µm	3 pcs	84,60
The state of the s	LOS80C-L/3	Ortho-Strips, One-Sided left, Extracoarse	80µm	3 pcs	84,60
	LOS80C-R/3	Ortho-Strips, One-Sided right, Extracoarse	80µm	3 pcs	84,60
	L071	Refill for IPR Set, Double-Sided		6 pcs	156,60
Strips System	L065DS	Ortho-Strips complete kit, Double-Sided		6 pcs	156,60
MA   Major	L065L	Ortho-Strips complete kit, One-Sided left		6 pcs	156,60
(cont)	L065R	Ortho-Strips complete kit, One-Sided right		6 pcs	156,60
	LOST400	Tray Ortho-Strips delivered with 6 silicone holders		1 pc	49,00
	D056ASS/6	Refills silicone holders		6 pcs	8,80

### **PRODUCTS PRICELIST**

REF. DESCRIPTION DIAMOND QUANTITY PRICES

GRIT-SIZE PACK IN FURO\*

## new ORTHO-STRIPS CENTRAL

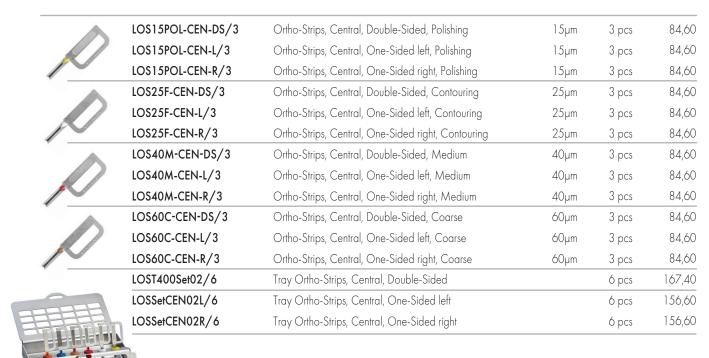

#### **IPR-DISTANCE CONTROL**



#### MANUAL INSTRUMENTS

